



The genuinely easy to use reading machine

When designing the Poet Compact 2 we focused on its main function: reading. This slender all-in-one reading machine can read virtually any kind of printed matter with just the press of a button. There is no need to study lengthy manuals to operate the Poet Compact 2. The built-in stereo loudspeakers directed towards the listener ensure a sound that is loud and clear. The controls are self explanatory, in tactile shape and have sensible pressing feedback as well as touch tones. Try it yourself – you'll be convinced of how easy it is to operate.

- Minimum controls
- Stand-alone, all-in-one unit
- Clear accurate speech
- Pleasing human-like inflection
- The 2-plus (2+) model comes with CD drive for DAISY, MP3, Audio CDs, etc. and advanced features
- Multiple languages on request
- Low energy consumption

Poet Compact 2 Poet Compact 2+

Poet Compact 2 with only 2 buttons and 2 control knobs

The Poet Compact 2 is the basic model with just 2 buttons. To use, lift the lid, place your printed text face down on the glass, and press the first button. This commands the Poet to scan the text for you and read it aloud using high quality speech. The second button is the pause button and stops and starts the speech immediately. The control knobs are used to adjust the volume and reading speed to your preference.

The Poet Compact 2+ has a front panel that offers more controls and possibilities: access to menu functions, speech settings, plug-ins, CD-drive for listening to DAISY-books, etc.

Technical specifications

Models:

- Poet Compact 2 with 2 buttons, 1 on/off button and 2 control knobs.
- Poet Compact 2+ with 3 buttons, 1 menu keypad, 1 on/off button and 2 control knobs

Scanner and OCR:

- Integrated flatbed scanner
- Modern speech synthesizer

Operation:

- On/off button, intuitive operation
- Tactile buttons with sensible pressing click feedback and touch tones
- Clearly arranged menu keypad

Formats:

- Audio CD, MP3, DAISY, TXT, RTF, DOC

Voltage:

- Universal power supply (100 - 240 V, 50 - 60 Hz)

Size and weight:

- 28 × 46 × 10 cm (appr. 11 × 18.1 × 3.9 in.)
- 5,8 kg (appr. 12.8 lbs.)



The Poet Compact 2 with 2 buttons.



Reading by means of one-touch operation – with 2 buttons or with 3 buttons and menu keypad.



Books, periodicals, magazines can be easily put onto the reading machine.

VisioBraille GmbH

Oßmaritzer Straße 4c phone.: 03641 - 2816-400
D-07745 Jena fax.: 03641 - 2816-416

e-mail: info@visiobraille.de
web: www.visiobraille.de

